


Transmitting 12 Lead EKG's


- Perform normal 12 lead EKG procedure
- After capture – envelope icon will show on left hand side. Select using soft key

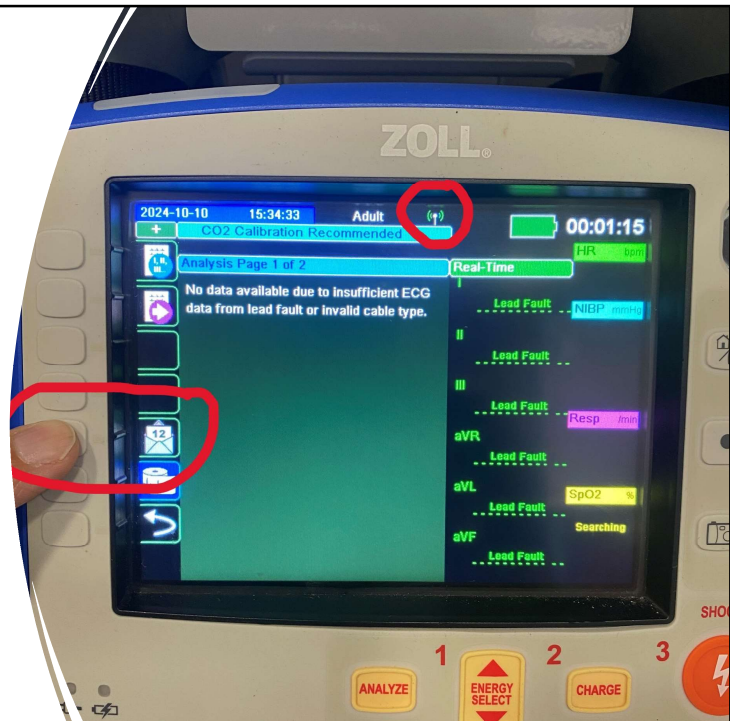

11

Transmitting 12 Lead EKG's


- Select the location you want 12 lead transmitted to
 - This can be both CCP and a hospital at the same time if you are doing a rendezvous
- Highlight and select "transmit"

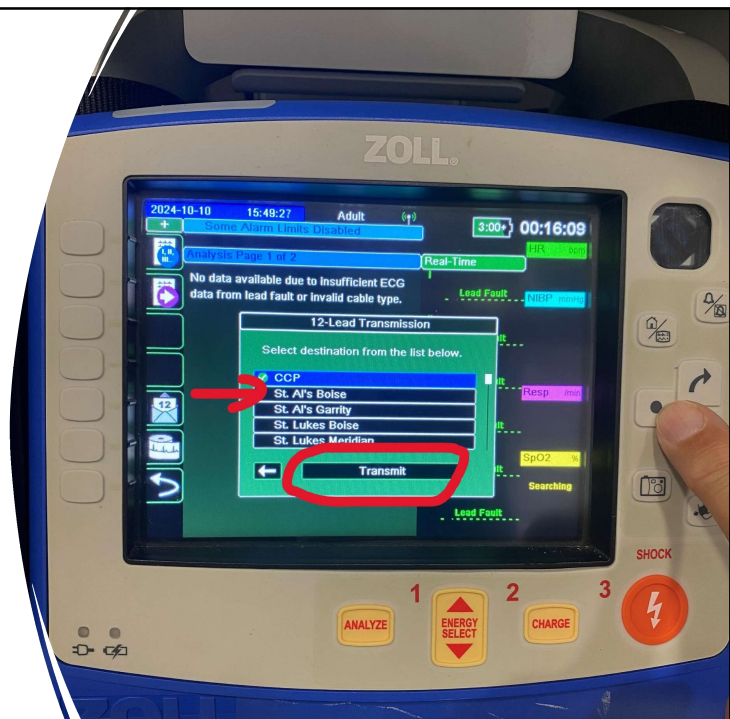

12


Transmitting 12 Lead EKG's

Look for the “transmission complete”
confirmation at the top of the screen

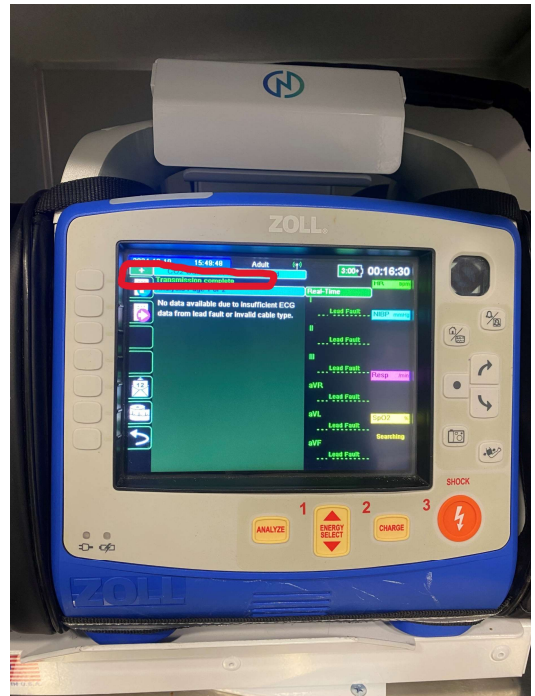

Transmitting 12 Lead EKG's


- Perform normal 12 lead EKG procedure
- After capture – envelope icon will show on left hand side. Select using soft key

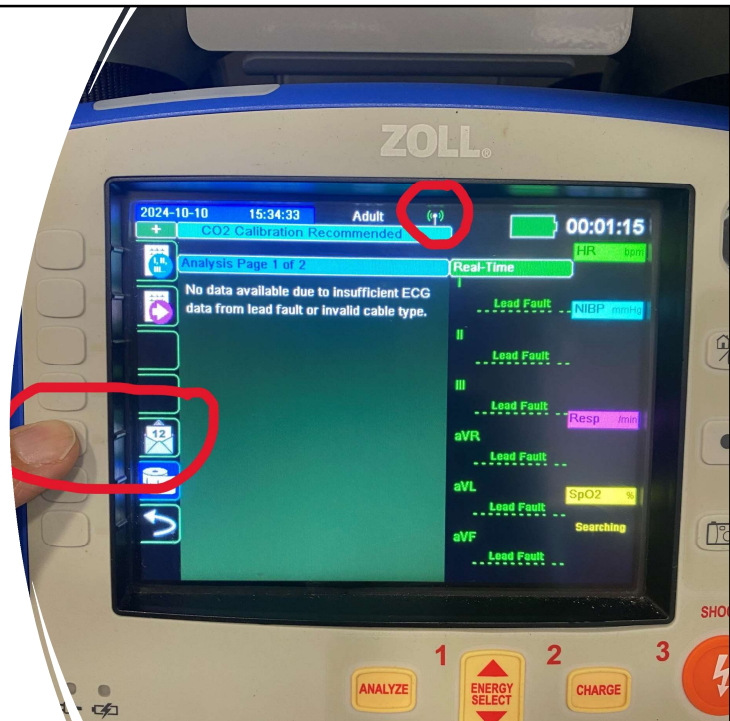

11

Transmitting 12 Lead EKG's

- Select the location you want 12 lead transmitted to
 - This can be both CCP and a hospital at the same time if you are doing a rendezvous
- Highlight and select "transmit"

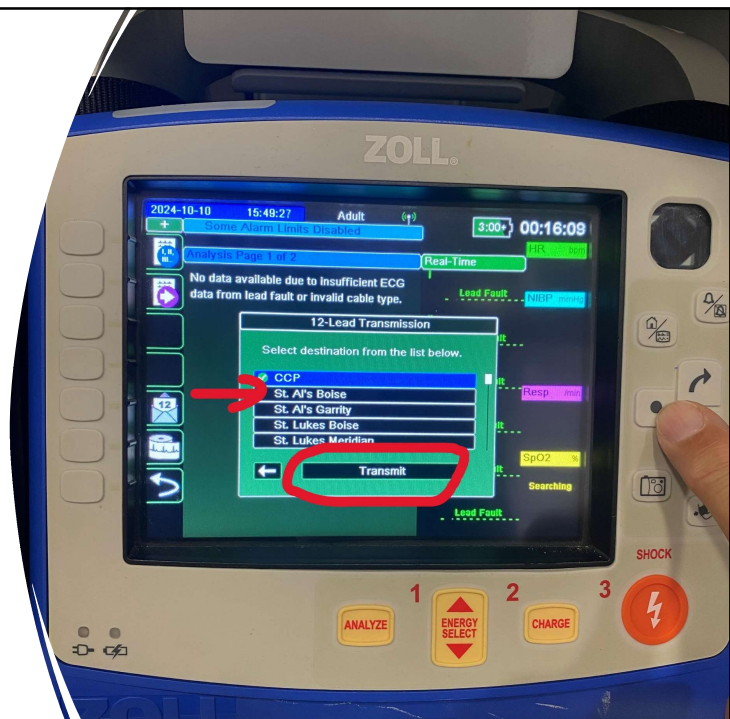

12

Transmitting 12 Lead EKG's

Look for the “transmission complete”
confirmation at the top of the screen

